

DIRECTIONS FOR USE
ART. S3041
BACK BRACE WITH POWER STRAPS

- Brace provides support of the back muscles and stabilizes abdominal pressure for safer activity.
- Reduces loads on the vertebrae and intervertebral discs.
- Aeroprene provides heating and micro massage action, non-blocking skin respiration.
- Tapered shape for a comfortable fit.
- Two plastic stays in back provide firm support to lumbar area.
- Reinforced strap design for adjustable compression.
- Removable massage pad in lower back area stimulates back muscles.
- Universal size (one size fits most).

Back brace with power straps features stays and power straps. It is designed to help reduce unwanted or excessive movements of the lower back. It is recommended in case of chronic back ache as well as for repetitive sporting or occupational activities. The variable compression fastenings ensure a close, comfortable fit and the removable insert is designed to help provide a soothing lumbar massage during activities, aiding with muscular spasm.

The support can be used for curing of osteochondrosis, radiculitis, displacements and hernias of vertebral disks, traumas, chronic pain in lumbosacral part of spine. Due to the durable, flexible and breathable material and plastic stays the brace provides reliable support of the lumbar area, relaxation of back muscles, and normalization of blood circulation.

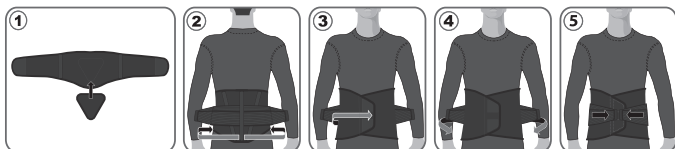
Recommendations for use:

Before use please consult with your doctor. Support is recommended to wear directly on underwear. Individual treatment and terms are determined by the physician.

Contraindications:

Skin local diseases, individual intolerance of the raw material components.

Dimensions	Sizes	Max deviation
	UNIVERSAL SIZE	
Waist circumference, cm	70 - 110	± 1,0 cm

How to get it on:

Maintenance:

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

Precautions:

Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

Contents: aeroprene – 50%, polyester – 25%, cotton - 15%, latex – 5%, kapron - 5%, plastic splints, foam pad.

Storage conditions: store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

Manufacturing date: see on packaging.

Size: universal size.

Medtex Swiss Ltd, 15 Konservna str., Gabrovo, Bulgaria 5300

tel.: +359 888 059 356, e-mail: info@medtex-swiss.ch,

medpack-group.com, medtex-swiss.ch, dr-frei.com

Dr. Frei is a registered Trademark of FreiCare Swiss AG, Switzerland. A Swiss Brand.