

DIRECTIONS FOR USE

ART. S6010

KNEE PATELLA STRAP BRACE FASTENING

Application:

- Treatment and prevention of tendovaginitis, jumper's knee
- The pad redistributes bandage compression, provides massaging effect.
- The inner layer made of soft cotton fabric for comfort.
- Relieves pain of strained muscles during activities.
- Excellent fixation and compression due to additional non-stretching strap with velcro fastener.
- Simple design, comfortable for wearing without assistance.
- Universal size (one size fits most).
- Fits both left and right knees.

Indications for use:

Knee brace is specifically designed to help provide dynamic compression and support to knee joint complex. The vibration dampening pad helps ensures additional protection and support during occupational and sporting activities. It is useful in cases of jumper's knee, reducing tendon irritation and overuse. The knee brace removes the weight from knee joint.

- It is recommended in cases of chronic overloads of the knee joint (sports, professional), trauma, flat feet, patellofemoral pain syndrome, Osgood-Schlatter disease.
- Anatomically shaped bandage ensures correct position during activities.


Recommendations for use:

Before use please consult with your doctor. Support is recommended to wear directly on underwear. Individual treatment and terms are determined by the physician.

Contraindications:

Skin local diseases, individual intolerance of the raw material components.





Dimensions	Universal size	Max deviation
Knee circumference, cm	30-43	± 1,0 cm

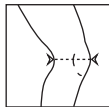

How to get it on:

for left leg

for right leg

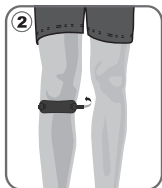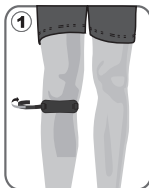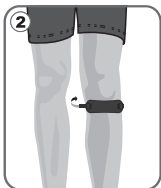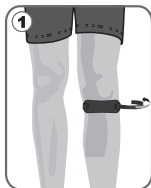

Maintenance:

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

Precautions:

Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

Contents: aeroprene - 60%, polyester - 25%, cotton - 15%.

Storage conditions: store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

Manufacturing date: see on packaging.

Size: universal size.

Medtex Swiss Ltd, 15 Konservna str., Gabrovo, Bulgaria 5300

tel.: +359 888 059 356, e-mail: info@medtex-swiss.ch,

medpack-group.com, medtex-swiss.ch, dr-frei.com

Dr. Frei is a registered Trademark of FreiCare Swiss AG, Switzerland. A Swiss Brand.