

**DIRECTIONS FOR USE**

**ART. S6040**

**KNEE JOINT SUPPORT ELASTIC**

**Application:**

- Provides reliable support and protection of knee joint during activities.
- Heat retention and compression reduce risk of strains or injuries.
- The anatomical shape is convenient for daily use.
- Breathable fabric ensures additional comfort.
- Fits both left and right knees.

**Indications for use:**

Knee support ensures dynamic support and adjustable compression to the knee joint without restricting movement. It helps distribute the load over the joint during the movement, so relieves arthritic and meniscus pain.

Inflammatory and degenerative diseases of the knee joint (initial stages of arthritis, synovitis, periarthritic edema, bursitis or arthrosis). Prevention of knee and/or kneecap trauma during participation in sports and physical activities. Chronic instability of the knee cap (recurrent subluxation). Injuries and ligamentous laxity of the knee joint in subacute phase or midterm and late rehabilitation after removing a plaster cast.

- It is recommended in case of arthritic knees, chronic aches and stiffness.
- It helps in case of strains, sprains, instability and patellar tracking.
- Provides support and stabilizes injured, weak or arthritic knees during occupational and sporting activities.
- Increases healing and keeps joints and muscles flexible.

**Recommendations for use:**

Before use please consult with your doctor. Support is recommended to wear directly on underwear. Individual treatment and terms are determined by the physician.

**Contraindications:**

Skin local diseases, individual intolerance of the raw material components.





Dimensions	Sizes				Max deviation
	S	M	L	XL	
Knee circumference, cm	28-33	34-39	40-45	46-51	± 1,0 cm



**How to get it on:**

for left leg

for right leg

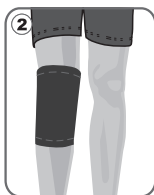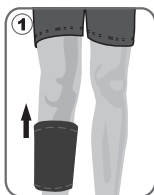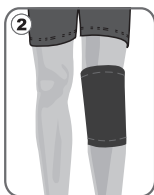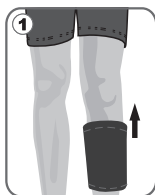

**Maintenance:**

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

**Precautions:**

Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

**Contents:** capron - 60%, latex - 30%, polyester - 10%.

**Storage conditions:** store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

**Manufacturing date:** see on packaging.

**Size:** see on packaging.

Medtex Swiss Ltd, 15 Konservna str., Gabrovo, Bulgaria 5300  
tel.: +359 888 059 356, e-mail: info@medtex-swiss.ch,  
medpack-group.com, medtex-swiss.ch, dr-frei.com

Dr. Frei is a registered Trademark of FreiCare Swiss AG, Switzerland. A Swiss Brand.