

**DIRECTIONS FOR USE**
**ART. S6058**
**STABILIZING KNEE SUPPORT WITH 4 SPIRAL STAYS**

- Bandage design ensures adjustable fit and unrestricted range of motion.
- Patellar buttress helps stabilize and protect kneecap.
- Additional circle pad around patella provides protection and stabilization to prevent patella displacement.
- Two medial and lateral spiral removable stays provide additional support and stability.
- Heat retention and compression reduce the risk of strains or injuries.
- Contour design and adjustable straps ensure stability and prevent slipping. Provides easy application and maximum support.
- Breathable aeroprene material lined with cotton behind knee for optimal comfort. Provides warmth and equal compression to weak or overstressed knees.
- Universal size (one size fits most).
- Fits both left and right knees.

Stabilizing knee support with 4 spiral stays ensures dynamic support and adjustable compression to the knee joint without restricting movement. The open patellar buttress fixes the kneecap and provides additional support and overall stability of the knee joint. It helps distribute the load over the joint during the movement, so relieves arthritic and meniscus pain.

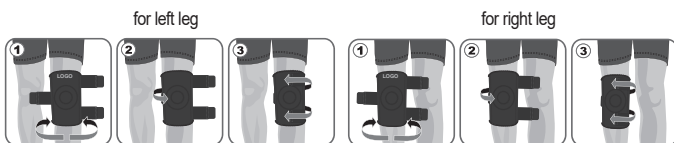
Inflammatory and degenerative diseases of the knee joint (initial stages of arthritis, synovitis, periarticular edema, bursitis or arthrosis), if warming knee support is not contraindicated. Prevention of knee and/or kneecap trauma during participation in sports and physical activities. Chronic instability of the kneecap (recurrent subluxation). Injuries and ligamentous laxity of the knee joint in subacute phase or midterm and late rehabilitation after removing a plaster cast.

- It is recommended in case of arthritic knees, chronic aches and stiffness.
- It helps in case of strains, sprains, instability and patellar tracking.
- Provides support and stabilizes injured, weak or arthritic knees during occupational and sporting activities.
- Increases healing and keeps joints and muscles flexible.

**Contraindications:**

Skin local diseases, individual intolerance of the raw material components.

Dimensions	Sizes	Max deviation
	UNIVERSAL SIZE	
Knee circumference, cm	30 - 41	± 1,0 cm

**How to get it on:**

**Maintenance:**

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

**Precautions:**

Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

**Contents:** cotton - 15%, aeroprene - 60%, polyester - 20%, latex - 5%, metallic spiral splints.

**Storage conditions:** store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

**Manufacturing date:** see on packaging.

**Size:** universal size.

Medtex Swiss Ltd, 15 Konservna str., Gabrovo, Bulgaria 5300

tel.: +359 888 059 356, e-mail: info@medtex-swiss.ch,

medpack-group.com, medtex-swiss.ch, dr-frei.com

Dr. Frei is a registered Trademark of FreiCare Swiss AG, Switzerland. A Swiss Brand.