

## DIRECTIONS FOR USE

**ART. S8514**

### WRIST BAND ADJUSTABLE

#### Product description:

- Wrist band relieves pain due to compression and fixation of the injured joint.
- Breathable elastic material provides moderate fixation and stabilization of the wrist joint.
- Provides therapeutic heat and compressive support to wrist joint.
- Adjustable velcro strap customizes compression around wrist and ensures comfortable fit and stabilization during activities.
- The inner layer made of soft cotton fabric for comfort.
- Universal size (one size fits most).
- Fits both left and right hands.

#### Indications for use:

Pain affecting wrist caused by trauma or injury and different wrist joint disorders. Limited range of motion caused by trauma or injury and by ligamentous laxity of the wrist joint. Chronic tendon inflammation;

Condition after trauma and/or operation. Professional hazardous exposures on the wrist (vibration, rotation, lifting of weights). Trauma prevention at all levels of sport and physical activities.

- Breathable material (aeroprene) keeps warmth not interfering with skin functions (protection, thermoregulation etc), improves blood circulation and ensures maximal comfort
- Elastic loop provides additional comfort when putting on
- The bandage fits both left and right hands

#### Contraindications:

Skin local diseases, individual intolerance of the raw material components.

Dimensions	Sizes	Max deviation
	UNIVERSAL SIZE	
Wrist circumference, cm	14 - 24	± 1,0 cm

#### How to get it on:

for left hand



for right hand



#### Maintenance:

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

#### Precautions:

Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

**Contents:** cotton - 15%, aeroprene - 65%, polyester - 20%.

**Storage conditions:** store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

**Manufacturing date:** see on packaging.

**Size:** universal size.

 Medtex Swiss Ltd, 15 Konservna str., Gabrovo, Bulgaria 5300

tel.: +359 888 059 356, e-mail: [info@medtex-swiss.ch](mailto:info@medtex-swiss.ch),

[medpack-group.com](http://medpack-group.com), [medtex-swiss.ch](http://medtex-swiss.ch), [dr-frei.com](http://dr-frei.com)

Dr. Frei is a registered Trademark of FreiCare Swiss AG, Switzerland. A Swiss Brand.

DR. FREI SPORT IS A REGISTERED TRADEMARK.  
A SWISS BRAND.



REF DFS001\_091117

