

## DIRECTIONS FOR USE

**ART. S8551**

### STABILIZING WRIST BRACE ADJUSTABLE

- Stabilizing wrist brace features a removable metal splint for variable stabilization of the weak or stiff wrist in a neutral position.
- Support and protection for athletic and work activities.
- Provides reliable fixation and compression of the radiocarpal joint.
- Anatomically contoured metal stiffening plate and additional straps provide reliable stabilization of the joints and greater wrist stability.
- The inner layer made of soft cotton fabric for comfort.
- Universal size (one size fits most).
- Designed separately for left and right hands.

#### Indications for use:

Brace is used at wrist injuries, ligament sprains, arthrosis, arthritis, bursitis, synovitis, at rehabilitation stage after plaster removal as well as for prevention of arthritis, bursitis, tendovaginitis and their exacerbation.

It is intended for complete fixation of radiocarpal joint, provides warming and equal compression. The brace is strengthened with rigid metal insert that completely limits motions of the radiocarpal joint.

The adjustable strap provides additional compression around the wrist if needed which can help encourage correct alignment of the carpal bones and therefore to reduce inflammation in the wrist, which is useful in the management of carpal tunnel syndrome.

- Provides reliable fixation and compression of the radiocarpal joint.
- The metal stiffening plate and additional straps ensure reliable stabilization of the joint.
- Anatomic shape is comfortable for daily use


#### Contraindications:

Skin local diseases, individual intolerance of the raw material components.


| Dimensions              | Sizes          | Max deviation |
|-------------------------|----------------|---------------|
|                         | UNIVERSAL SIZE |               |
| Wrist circumference, cm | 15 - 22        | ± 1,0 cm      |

#### How to get it on:

for left hand

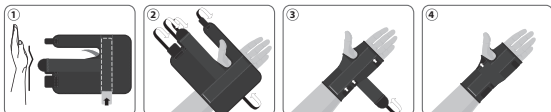

for right hand

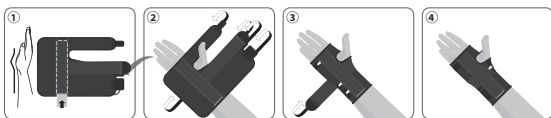

#### Maintenance:

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

#### Precautions:

Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

**Contents:** cotton - 15%, aeroprene - 60%, polyester - 20%, latex - 5%, metallic splint.

**Storage conditions:** store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

**Manufacturing date:** see on packaging.

**Size:** universal size.

Medtex Swiss Ltd, 15 Konservna str., Gabrovo, Bulgaria 5300

tel.: +359 888 059 356, e-mail: info@medtex-swiss.ch,

medpack-group.com, medtex-swiss.ch, dr-frei.com

Dr. Frei is a registered Trademark of FreiCare Swiss AG, Switzerland. A Swiss Brand.