



DIRECTIONS FOR USE

ART. S8503

WRIST WRAP ELASTIC

Application:

- Provides soft fixation and compression for wrist joint.
- . The anatomical shape is convenient for daily use
- Breathable fabrics stretches in four directions for more comfortable usage
- Fits both left and right hands.

Indications for use:

Can be used for prevention of common dislocations and traumas while physical exercises. Supports can also be used for trauma, sprain, arthrosis and arthritis, while rehabilitation after plaster cast.

Recommendations for use:

Before use please consult with your doctor. Support is recommended to wear directly on underwear. Individual treatment and terms are determined by the physician.


Contraindications:

Inflammatory skin disease in the area of brace application, individual intolerance to the brace materials.

Dimensions	Sizes					
	s	м	L	XL	Max deviation	A
Wrist circumference, cm	14-16	17-19	20-22	23-25	± 1,0 cm	

How to get it on:

for left hand

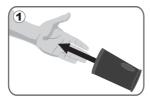



for right hand





Maintenance:

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorinecontaining bleaching detergents). Don't use dry-cleaning.

Precautions

Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

Contents: capron - 60%, latex - 30%, polyester - 10%.

Storage conditions: store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

Manufacturing date: see on packaging.

Size: see on packaging.

Medtex Świss Ltd, 15 Konservna str., Gabrovo, Bulgaria 5300

tel.: +359 888 059 356, e-mail: info@medtex-swiss.ch,

medpack-group.com, medtex-swiss.ch, dr-frei.com

Dr. Frei is a registered Trademark of FreiCare Swiss AG, Switzerland. A Swiss Brand.

DR. FREI IS A REGISTERED TRADEMARK. A SWISS BRAND.





